

Recommended protocol for TdT

Obtained in run 52

12 Dec 2017

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: SEN28
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-TdT-339 / 6032592
Diluent: Antibody Diluent
Diluent producer / no.: Ventana / 251-018
Dilution factor: 1:25
Incubation time / temperature: 60 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.