

Recommended protocol for CGA

Obtained in run 53

07 Mar 2018

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: LK2H10+PHE5
Producer: NeoMarkers
Product no. / lot no.: MS-382-P1 / 382P1505H
Diluent: Antibody Dilution Buffer
Diluent producer / no.: Ventana / ADB250
Dilution factor: 1:800
Incubation time / temperature: 40 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 24 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 37°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.