

Recommended protocol for CGA

Obtained in run 53

13 Mar 2018

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: LK2H10+PHE5
Producer: Thermo Fisher Scientific
Product no. / lot no.: MA5-13287 / TB2524213A
Diluent: Bond Antibody Diluent
Diluent producer / no.: Leica / AR9352
Dilution factor: 1:200
Incubation time / temperature: 15 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Leica
Product / no: Bond Polymer Refine Red Detection / DS9390
Incubation time linker: 20 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Polymer Refine Red Detection / DS9390
Incubation time / temperature: 15 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.