

Recommended protocol for OCT3/4

Obtained in run 53

14 Mar 2018

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: N1NK
Producer: Dako/Agilent
Product no. / lot no.: M3649 / 10134329
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:50
Incubation time / temperature: 20 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 25°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.