

## Recommended protocol for WT1

Obtained in run 55

02 Jan 2019

### Immunostainer

Type: Tissue-Tek Genie Advanced Stai

### Primary antibody

Clone: EP122  
Producer: Epitomics  
Product no. / lot no.: AC-0115 / CI051909L  
Diluent: Genie Primary Antibody Diluent 2.0  
Diluent producer / no.: Sakura Finetek / GS-21018  
Dilution factor: 1:400  
Incubation time / temperature: 30 min. / 25°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Sakura Finetek Tissue-Tek Genie High pH Antigen Retrieval  
Heating time at max. temp.: 45 min.  
Maximum heating temp.: 98°C

### Visualization system

Producer: Sakura Finetek  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Linker: Tissue-Tek Genie Pro Detection Kit, DAB  
Incubation time linker: 15 min.  
Incubation time polymer: 21 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Sakura Finetek  
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.