

Recommended protocol for PD-L1 (uro)

Obtained in run C5

26 Feb 2019

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: 22C3
Producer: Dako
Product no. / lot no.: M365329-2 / 10143018
Diluent: Bond Antibody Diluent
Diluent producer / no.: Leica / AR9352
Dilution factor: 1:10
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 25 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.