

Recommended protocol for CD117

Obtained in run 56

27 Feb 2019

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: YR145
Producer: Cell Marque
Product no. / lot no.: 117R-14-ASR , 117R-1 / 1617503e
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:250
Incubation time / temperature: 20 min. / 23°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 10 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8002/SM802
Linker: Linker, Rabbit
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 23°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 23°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.