

Recommended protocol for CK-LMW

Obtained in run

20 Sep 2015

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: CAM 5.2
Producer: Becton Dickinson
Product no. / lot no.: 345779 / 29678
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 32 min. / 37°C

Epitope retrieval, proteolysis

Enzyme: Protease 1
Enzyme producer / no: Ventana / 760-2018
Incubation time / temp: 8 min. / 37°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universa DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.