

Recommended protocol for WT1

Obtained in run

23 Sep 2015

Immunostainer

Type: IntelliPath, Biocare

Primary antibody

Clone: 6F-H2
Producer: Dako
Product no. / lot no.: M3561 / 0093F
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 5 min.
Maximum heating temp.: 103°C

Visualization system

Producer: Immunologic
Product / no: NS / DPVB-110HRP
Linker: NS
Incubation time linker: 15 min.
Incubation time polymer: 25 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 5 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.