

Recommended protocol for CD68

Obtained in run 60

03 Aug 2020

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: KP1
Producer: Dako
Product no. / lot no.: M0814 / 20058801
Diluent: Da Vinci Green
Diluent producer / no.: Biocare Medical / PD900M
Dilution factor: 1:4000
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 32 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 22°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.