

Recommended protocol for CD31

Obtained in run 62

05 Mar 2021

Immunostainer

Type: Tissue-Tek Genie

Primary antibody

Clone: RM247
Producer: SAKURA
Product no. / lot no.: 8282-C010 / 8282A28320
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: SAKURA Tissue-Tek High pH
Heating time at max. temp.: 45 min.
Maximum heating temp.: 97°C

Visualization system

Producer: SAKURA
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250
Incubation time linker: 15 min.
Incubation time polymer: 10 min.
Incubation temperature: 25°C

Chromogen

Producer: 8826-K250
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / SAKURA
Incubation time / temperature: 8 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.