

Recommended protocol for CK20

Obtained in run 62

15 Mar 2021

Immunostainer

Type: Sakura Genie

Primary antibody

Clone: Ks 20.8
Producer: Sakura
Product no. / lot no.: 8304 / 8304A34620
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Sakura Sakura Finetek USA High pH Antigen Retrieval solution
Heating time at max. temp.: 45 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Sakura
Product / no: Tissue-Tek Genie Pro kit / 8826-K250
Incubation time polymer: 20 min.
Incubation temperature: 21°C

Chromogen

Producer: Sakura
Product / no: DAB / 8826-K250
Incubation time / temperature: 6 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.