

Recommended protocol for CD79a

Obtained in run 63

09 Aug 2021

Immunostainer

Type: CNT330

Primary antibody

Clone: C4C4
Producer: Celnovte
Product no. / lot no.: CCM-0734 / 202106201
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Celnovte
Product / no: CNTVision Super PolyDetection / SD5100
Linker: CNTVision Super PolyDetection
Incubation time linker: 20 min.
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Celnovte
Product / no: CNTVision Super PolyDetection / SD5100
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.