

Recommended protocol for MSA

Obtained in run

26 Sep 2015

Immunostainer

Type: IntelliPath, Biocare

Primary antibody

Clone: HMB-45
Producer: Dako
Product no. / lot no.: M0634 / 1075695
Diluent: Antibody Diluent
Diluent producer / no.: Nordic Biosite / NS
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare Diva Diloaker pH 6,2
Heating time at max. temp.: 5 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare
Product / no: NS / M4U534L
Linker: NS
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 20°C

Chromogen

Producer: Biocare
Product / no: DAB / M4U534L
Incubation time / temperature: 5 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.