

Recommended protocol for TdT

Obtained in run

26 Sep 2015

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: Polyclonal
Producer: Immunologic
Product no. / lot no.: ILP 0049 C1 / 1021
Diluent: Antibody Diluent
Diluent producer / no.: Sytec / NS
Dilution factor: 1:200
Incubation time / temperature: 45 min. / 20°C

Epitope retrieval, HIER

Device: Water bath
Buffer: Klinipath TRIS/EDTA pH 9
Heating time at max. temp.: 60 min.
Maximum heating temp.: 90°C

Visualization system

Producer: BrightVision
Product / no: NS / DPVR-500HRP
Incubation time polymer: 25 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 5 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.