

Recommended protocol for CK-LMW

Obtained in run

26 Sep 2015

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: C51
Producer: Invitrogen / Zymed
Product no. / lot no.: 18-185Z / 762348A
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S0809
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 30 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Dako
Product / no: EnVision+ System- HRP / K4001
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / K3467 / K3468
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.