

## Recommended protocol for CD56

Obtained in run 64

10 Jan 2022

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: MRQ-42  
Producer: Cell Marque  
Product no. / lot no.: 156R-94/95/96 / 0000122883  
Diluent: Antibody Dilution Buffer  
Diluent producer / no.: Ventana / ADB250  
Dilution factor: 1:100  
Incubation time / temperature: 20 min. / 25°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: Linker, Mouse  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.