

Recommended protocol for Napsin A

Obtained in run 66

22 Aug 2022

Immunostainer

Type: Intellipath Biocare MEdical

Primary antibody

Clone: BC15
Producer: Biocare Medical
Product no. / lot no.: API3043AA / 011222A
Diluent: Da Vinci Green
Diluent producer / no.: Biocare Medical / PD900L
Dilution factor: 1:1
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Biocare MEdical
Product / no.: MACH-4UNIVERSAL HRP-POLYMER DETECTION SYSTEM
Linker: MACH4 Mouse Probe
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Amplifier: MACH4 HRP Polymer
Incubation time amplifier: NA min.
Incubation temperature: 25°C

Chromogen

Producer: Biocare Medical
Product / no.: Betazoid DAB chromogen / BDB900F
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided



Nordic Immunohistochemical Quality Control

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.