

Recommended protocol for CD4

Obtained in run 67

08 Dec 2022

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: EP204
Producer: Epitomics
Product no. / lot no.: AC-0173 / 230100562
Diluent: Renoir Red Diluent
Diluent producer / no.: Biocare Medical / PD904
Dilution factor: 1:25
Incubation time / temperature: 30 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: epreDia Dewax and HIER Buffer H
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Rabbit LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 36°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 36°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.