

Recommended protocol for MLH1

Obtained in run 67

10 Jan 2023

Immunostainer

Type: Leica BOND Max

Primary antibody

Clone: IHC409
Producer: GenomeMe
Product no. / lot no.: IHC409 / C40912051
Diluent: GenomeMe Antibody Diluent
Diluent producer / no.: GenomeMe / IHC000
Dilution factor: 1:50
Incubation time / temperature: 15 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.