

## Recommended protocol for PSA

Obtained in run 69

04 Jul 2023

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: ER-PR8  
Producer: Dako/Agilent  
Product no. / lot no.: M0750 / 41481190  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:50  
Incubation time / temperature: 20 min. / 32°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.