

Recommended protocol for CD138

Obtained in run 69

04 Jul 2023

Immunostainer

Type: CNT360

Primary antibody

Clone: C5G6
Producer: Celnovte
Product no. / lot no.: CCM-0782 / 202304003
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 20 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Celnovte
Product / no: MicroStacker TM PLUS / SD5600
Linker: MicroStacker TM PLUS
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 25°C

Chromogen

Producer: Celnovte
Product / no: MicroStacker TM PLUS / SD5600
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.