

Recommended protocol for CD138

Obtained in run 69

11 Jul 2023

Immunostainer

Type: Qpath 48

Primary antibody

Clone: BY083
Producer: Bioin Biotechnology
Product no. / lot no.: BFM-0397 / 20220801
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:Ready-To-U
Incubation time / temperature: 30 min. / 37°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 22 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Bioin Biotechnology
Product / no: DAB Detection Kit / BFM-0309
Incubation time polymer: 10 min.
Amplifier: DAB Detection Kit
Incubation time amplifier: 8+8 min.
Incubation temperature: 37°C

Chromogen

Producer: Bioin Biotechnology
Product / no: DAB Detection Kit / BFM-0309
Incubation time / temperature: 6 min. / 37°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.