

## Recommended protocol for CD138

Obtained in run 69

11 Oct 2023

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: B-A38  
Producer: Immunologic  
Product no. / lot no.: ILM3825-C1 / 060721-A  
Diluent: Bright diluent  
Diluent producer / no.: Immunologic / UD09-999  
Dilution factor: 1:400  
Incubation time / temperature: 60 min. / 25°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Immunologic  
Product / no: Brightvision / DPVB999HRP-polymer  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Immunologic  
Product / no: Bright Dab Solution / UBVB999HRP-BS04A  
Incubation time / temperature: 8 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.