

Recommended protocol for EpCAM

Obtained in run 69

16 Aug 2023

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: BS14
Producer: Nordic Biosite
Product no. / lot no.: BSH-7402-100 / BSH-7402-1
Diluent: Bond Antibody Diluent
Diluent producer / no.: Leica / AR9352
Dilution factor: 1:100
Incubation time / temperature: 20 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Leica
Product / no: Bond Polymer Refine Red Detection / DS9390
Incubation time linker: 20 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Polymer Refine Red Detection / DS9390
Incubation time / temperature: 15 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.