

Recommended protocol for CD5

Obtained in run 69

31 Aug 2023

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: 4C7
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-CD5-4C7 / 6093334
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S3022
Dilution factor: 1:100
Incubation time / temperature: 60 min. / 20°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: None
Incubation time polymer: 25 min.
Incubation temperature: 20°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.