

Recommended protocol for Bcl-6

Obtained in run 70

13 Feb 2024

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: E5I8I
Producer: Cell Signaling
Product no. / lot no.: 89369S / 1
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: epredia Dewax and HIER Buffer H
Heating time at max. temp.: 24 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Dual LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 36°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 36°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.