

Recommended protocol for Bcl-6

Obtained in run 70

09 Jan 2024

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: LN22
Producer: Epredia
Product no. / lot no.: MS-1936-S / 1936P0523
Diluent: Bright diluent
Diluent producer / no.: Immunologic / UD09-999
Dilution factor: 1:100
Incubation time / temperature: 60 min. / 25°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Immunologic
Product / no: Brightvision / UBVB999HRP-polymer
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 25°C

Chromogen

Producer: Immunologic
Product / no: Bright DAB / UBVB999HRP-BSO4B
Incubation time / temperature: 8 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.