

Recommended protocol for CGA

Obtained in run 70

10 Jan 2024

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: LK2H10
Producer: Thermo Scientific
Product no. / lot no.: MS-324-P / 324P1810C
Diluent: Bond Antibody Diluent
Diluent producer / no.: Leica / AR9352
Dilution factor: 1:800
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 30 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.