

Recommended protocol for CK19

Obtained in run 50

14 Mar 2017

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: RCK108
Producer: Dako
Product no. / lot no.: M0888 / 20015577
Diluent: Antibody Diluent
Diluent producer / no.: Dako / S3022
Dilution factor: 1:50
Incubation time / temperature: 39 min. / 23°C

Epitope retrieval, proteolysis

Enzyme: Protease XIV
Enzyme producer / no: SIGMA / P5147-5G
Incubation time / temp: 20 min. / 37°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 23°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 15 min. / 23°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.