

Recommended protocol for TRPS1

Obtained in run 72

04 Jul 2024

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: ZR382
Producer: Zeta Corporation
Product no. / lot no.: Z2673RL / ZR04212022B-4
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:100
Incubation time / temperature: 25 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 30 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.