

Recommended protocol for p16

Obtained in run 72

08 Jul 2024

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: MX007
Producer: vitro master diagnostic ITK
Product no. / lot no.: MAD-000690Q / 0690Q134
Diluent: Renoir Red Diluent
Diluent producer / no.: Biocare Medical / PD904
Dilution factor: 1:100
Incubation time / temperature: 15 min. / 21°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica BOND-PRIME Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 21°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 21°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.