

Recommended protocol for CD8

Obtained in run 72

12 Aug 2024

Immunostainer

Type: Gene Tech GAS

Primary antibody

Clone: SP16
Producer: Gene Tech
Product no. / lot no.: GT211202 / 2024073101
Diluent: None
Diluent producer / no.: (None) / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 30°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Gene Tech
Product / no: DAB Dectection kit (For GAS) / GK8010
Linker: DAB Dectection kit (For GAS)
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 30°C

Chromogen

Producer: Gene Tech
Product / no: DAB Dectection kit (For GAS) / GK8010
Incubation time / temperature: 7 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.