

Recommended protocol for ALK (lung)

Obtained in run 73

26 Nov 2024

Immunostainer

Type: Dako Autostainer

Primary antibody

Clone: OTI1A4
Producer: Origene
Product no. / lot no.: TA801287 / F004
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:1000
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 10 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX / K8000/SM802
Linker: Linker, Mouse
Incubation time linker: 30 min.
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.