

Recommended protocol for Bcl-2

Obtained in run 73

05 Dec 2024

Immunostainer

Type: Celnovte CNT360

Primary antibody

Clone: C7B8
Producer: Celnovte
Product no. / lot no.: CBM-0041 / 202410001
Diluent: None
Diluent producer / no.: (None) / None
Dilution factor: 1:0
Incubation time / temperature: 20 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Celnovte
Product / no: MicroStacker TM PLUS Polymer Detection Kit / SD5600
Linker: MicroStacker TM PLUS Polymer Detection Kit
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 25°C

Chromogen

Producer: Celnovte
Product / no: MicroStacker TM PLUS Polymer Detection Kit / SD5600
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.