

Recommended protocol for Bcl-2

Obtained in run 73

23 Dec 2024

Immunostainer

Type: Gene Tech GAS

Primary antibody

Clone: 124
Producer: Gene Tech
Product no. / lot no.: GM088702 / 2024111201
Diluent: None
Diluent producer / no.: (None) / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Gene Tech
Product / no: DAB Dectection kit (For GAS) / GK8010
Linker: DAB Dectection kit (For GAS)
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 25°C

Chromogen

Producer: Gene Tech
Product / no: DAB / GK8010
Incubation time / temperature: 7 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.