

Recommended protocol for MLA

Obtained in run 74

07 Feb 2025

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: EP43
 Producer: Nordic Biosite
 Product no. / lot no.: AC-0041 / -
 Diluent: REAL Antibody Diluent
 Diluent producer / no.: Dako / S2022
 Dilution factor: 1:25
 Incubation time / temperature: 20 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
 Buffer: Ventana Ultra CC1
 Heating time at max. temp.: 36 min.
 Maximum heating temp.: 100°C

Epitope retrieval, proteolysis

Enzyme: Protease 3
 Enzyme producer / no: Ventana / 760-2020
 Incubation time / temp: 4 min. / 36°C

Visualization system

Producer: Ventana
 Product / no: UltraView Universal Alkaline Phosphatase Red Detection Kit
 Incubation time polymer: 8 min.
 Amplifier: Ultraview/iView Amplification Kit
 Incubation time amplifier: 8+8 min.
 Incubation temperature: 36°C

Chromogen

Producer: Ventana
 Product / no: ultraView Universal Alkaline Phosphatase Red Detection Kit
 Incubation time / temperature: 20 min. / 36°C
 Enhancement: None



Nordic Immunohistochemical Quality Control

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.