

Recommended protocol for MLA

Obtained in run 74

12 Feb 2025

Immunostainer

Type: Ventana Benchmark GX

Primary antibody

Clone: A103
Producer: Dako
Product no. / lot no.: M7196 / 20017531
Diluent: REAL Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:25
Incubation time / temperature: 30 min. / 37°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 4 min.
Incubation time polymer: 4 min.
Amplifier: OptiView Amplification Kit
Incubation time amplifier: 4+4 min.
Incubation temperature: 37°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 37°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



Nordic Immunohistochemical Quality Control

Department of Pathology, Aalborg University Hospital, Hospitalsbyen 5, DK-9260 Gistrup, Denmark

the most recent developments in immunohistochemistry at any point in time.