

Recommended protocol for MLA

Obtained in run 74

13 Feb 2025

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: EP43
Producer: EPITOMICS
Product no. / lot no.: AC-0041A / 230101197
Diluent: Diamond - Antibody Diluent
Diluent producer / no.: Cell Marque / 938B
Dilution factor: 1:100
Incubation time / temperature: 15 min. / 22°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 2
Heating time at max. temp.: 30 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 21°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.