

Recommended protocol for MLA

Obtained in run 74

04 Apr 2025

Immunostainer

Type: Sakura Tissue-Tek Genie

Primary antibody

Clone: EP43
Producer: Sakura FineTek
Product no. / lot no.: 8319-C010 / 8319E04125
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Sakura Tissue-Tek Genie High pH Antigen Retrieval
Heating time at max. temp.: 60 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Sakura
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 20°C

Chromogen

Producer: Sakura
Product / no: Tissue-Tek Genie Pro Detection Kit, DAB / 8826-K250
Incubation time / temperature: 6 min. / 25°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.