

Recommended protocol for CyD1

Obtained in run 75

26 Jun 2025

Immunostainer

Type: Leica BOND Prime

Primary antibody

Clone: EP12
Producer: Master Diagnostica
Product no. / lot no.: MAD-000630Q / 0630Q003
Diluent: Bond Primary Antibody Diluent
Diluent producer / no.: Leica / AR9352
Dilution factor: 1:50
Incubation time / temperature: 15 min. / 21°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica BOND-PRIME Epitope Retrieval Solution 2
Heating time at max. temp.: 20 min.
Maximum heating temp.: 104°C

Visualization system

Producer: Leica
Product / no: BOND-PRIME Polymer DAB Detection System / DS9284
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 21°C

Chromogen

Producer: Leica
Product / no: BOND-PRIME Polymer DAB Detection System / DS9284
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.