

Recommended protocol for SOX10

Obtained in run 75

13 Aug 2025

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: BS7
Producer: Nordic Biosite
Product no. / lot no.: BSH-7959-XXX / BSH1-26AJ
Diluent: Antibody Diluent
Diluent producer / no.: Ventana / 251-018
Dilution factor: 1:200
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 32 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.