

Recommended protocol for CDX2

Obtained in run 76

05 Dec 2025

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: EPR2764Y
Producer: Cell Marque
Product no. / lot no.: 235R-14, -15 or -16 / 0000214092
Diluent: Antibody Diluent
Diluent producer / no.: Ventana / 251-018
Dilution factor: 1:200
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Amplifier: Ultraview/iView Amplification Kit
Incubation time amplifier: 8+8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.