

Recommended protocol for PAX5

Obtained in run 76

18 Dec 2025

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: DAK-Pax5
Producer: Agilent Dako
Product no. / lot no.: M7307 / 41723077
Diluent: Renoir Red Diluent
Diluent producer / no.: Biocare Medical / PD904
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823/GV900
Linker: Mouse LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 36°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 5 min. / 36°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.