

Recommended protocol for EMA

Obtained in run 77

24 Mar 2026

Immunostainer

Type: Ventana Benchmark Ultra Plus

Primary antibody

Clone: E29
Producer: Dako
Product no. / lot no.: M0613 / 41690959
Diluent: REAL Antibody Diluent
Diluent producer / no.: Dako / S2022
Dilution factor: 1:50
Incubation time / temperature: 24 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 32 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.