

Recommended protocol for MUM1

Obtained in run 77

30 Mar 2026

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: MUM1p
Producer: Dako/Agilent
Product no. / lot no.: M7259 / 41653924
Diluent: Antibody Diluent with Background Reducing Component
Diluent producer / no.: Dako / S3022
Dilution factor: 1:50
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 95°C

Visualization system

Producer: Ventana
Product / no: UltraView Universal DAB Detection Kit / 760-500
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: ultraView Universal DAB Detection Kit / 760-500
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.