

Recommended protocol for MUM1

Obtained in run 77

31 Mar 2026

Immunostainer

Type: Ventana Benchmark Ultra Plus

Primary antibody

Clone: MUM1p
Producer: Dako/Agilent
Product no. / lot no.: M7259 / 41781760
Diluent: Antibody Diluent
Diluent producer / no.: Dako / K8006
Dilution factor: 1:100
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 48 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO4

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.